


Daily Care of your Voice Prosthesis

Provox® Brush

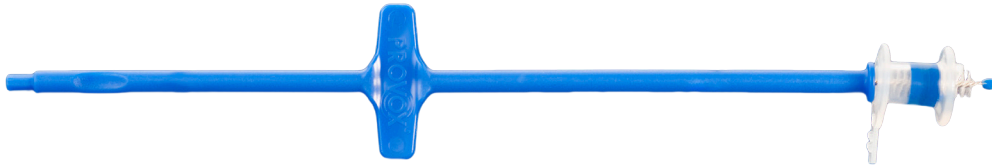

Clean the voice prosthesis in the morning and evening and after each meal using the Provox Brush and the Provox Flush.

1. Moisten the bristles of the Provox Brush and then insert all the way through the voice prosthesis. Pull back on the Brush if you feel any resistance.
2. Move the Brush back and forth while rotating the brush at the same time, this will clean the whole prosthesis.
3. When you are finished:
 - a. Clean the brush with drinking water and hand washing detergents.
 - b. Rinse with tap water and store dry.
4. Replace the Brush once a month or sooner if the bristles look worn and overused.

Tip: If needed the blue shaft of the brush can be bent for better access to the voice prosthesis. Do not bend the wired head of the Brush. Do not rotate the Brush during cleaning when the shaft is bent.

Tip: If your prosthesis is 12.5 mm or longer then use an XL Brush. If your prosthesis is 10 mm or shorter then use a regular Brush.

5. Disinfect the Provox Brush in accordance with the Provox Brush manual.

